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**[www.dryly.com](http://www.dryly.com)**

## TABLE OF CONTENT

Content of starter pack	<b>23</b>
Tips from the Dryly team	<b>24</b>
Installation	<b>25</b>
Receiver functions	<b>25</b>
Changing the Wi-Fi network	<b>28</b>
The evening ritual	<b>29</b>
Dryly webshop	<b>30</b>
Error messages	<b>31</b>
Warranty	<b>35</b>
Help and support	<b>35</b>

# CONTENT OF STARTER PACK

## THANK YOU FOR CHOOSING THE DRYLY BEDWETTING ALARM!

We are going to help you in the fight against bedwetting. Our goal is to help you as much as possible in the adventure to dry nights. We want your child to become motivated and enjoy taking on bedwetting.

In this manual we explain how to get started with the Dryly bedwetting alarm.

Dryly was founded because the son of co-founder Clarien had bedwetting problems for years. After years of trying all kinds of methods, her son came up with the idea of a fun, positive method. Why couldn't being potty-trained be fun? Why wasn't there a fun and motivating app? During Saturday morning breakfast the idea of Dryly was born.

Our vision is that we want to help both parents and children in the fight against bedwetting.

We are going on an adventure with you. An adventure is exciting, has a goal, but never has a straight track. We are at your side, with positive and fun products, but also with help, advice and tips.

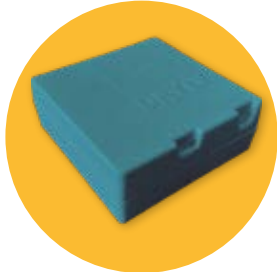
Underwear

Wizzu

Transmitter

Receiver

Charger

Mobile App  
(App Store or Googleplay Store)

## TIPS FROM THE DRYLY TEAM

- Keep in mind that this method takes an average of 3 months.

---

- Use the bedwetting alarm until your child has stayed dry for 14 nights.

---

- It is crucial to support your child during this period. Get out of bed too when your child wakes up from the alarm clock at night. Motivation and support are very important!

---

- Try to put as little pressure on your child as possible. Make it fun!

---

- The alarm stops going off by shaking Wizzu back and forth, so your child really needs to wake up.

---

- Besides your child being rewarded in the app with nice gadgets, it is nice to reward your child once in a while after a few dry nights.

---

- The transmitter cannot be charged. The battery lasts about one year.

---

- Don't wake up your child to pee in the evening.

---

- Save the bedwetting alarm; your child may have a setback.

---

- Be careful when washing the shorts, make sure you take the transmitter off. Transmitters cannot be put in the washing machine!

## INSTALLATION

### Before you get started:

1. Connect the receiver to the charger. This needs to load for at least 3 hours before it can be used.
2. Collect all parts from the starter pack.
3. Make sure you have your Wi-Fi password available.
4. Download the Dryly app in the Apple store or Google Play Store.
5. Install the Dryly app.
6. Create an account in the app.
7. Enter your child's information.

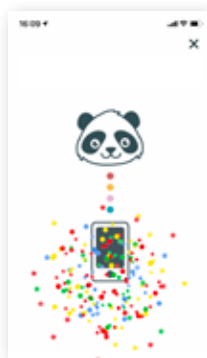
The installation will take a few minutes, we will help you through it! It is important to follow the steps of the app.

### 1 **Connect the receiver to the Wi-Fi network**

- Turn on the receiver only when the app says to do this!
- You hear a noise, this is the sign that it is working and the battery is charging.
- Connect to the "Wizzu" Wi-Fi network in the app.
- The app creates an overview of available networks.

- Select your home network and enter the password.
- When the receiver is connected correctly, you will hear a success sound.
- As soon as you see confetti in the app, the receiver has been successfully added!
- If connecting doesn't work please check if the permission settings regarding "location" are correct. Phone's settings › Dryly app › location › Make sure the check mark is set to "always".


## RECEIVER FUNCTIONS

- **Off**  
By switching the receiver to '0', you turn the receiver off.
- **Silent mode**  
In silent mode 'x' the receiver does not make any noise, but it does send a message to the Dryly app. Useful if you want to be a bit more discreet, during a sleepover for example.
- **On**  
By switching the switch to '1', you turn the receiver on.
- **Pairing mode**  
For the first three minutes, the receiver goes into pairing mode. When these three minutes have passed, the receiver connects to your home network. If this fails, or the receiver is not yet paired, it goes into 'offline' mode. You can use the bedwetting alarm, but you will not receive any messages in the Dryly app.

### 2 Connect transmitter to the receiver

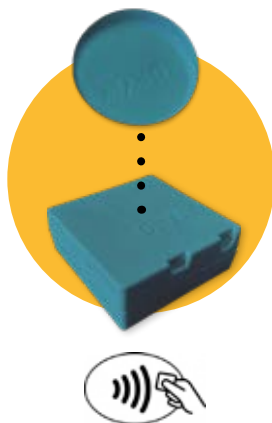
- Hold the transmitter with the Dryly logo side against the NFC square on the receiver.
- You hear "Transmitter connected".
- After a few minutes, the battery position becomes visible in the app. You may need to reload the app.

### 3 Dressing Wizzu

- Now that the bedwetting alarm has been installed successfully, your child can dress up Wizzu with the first points.

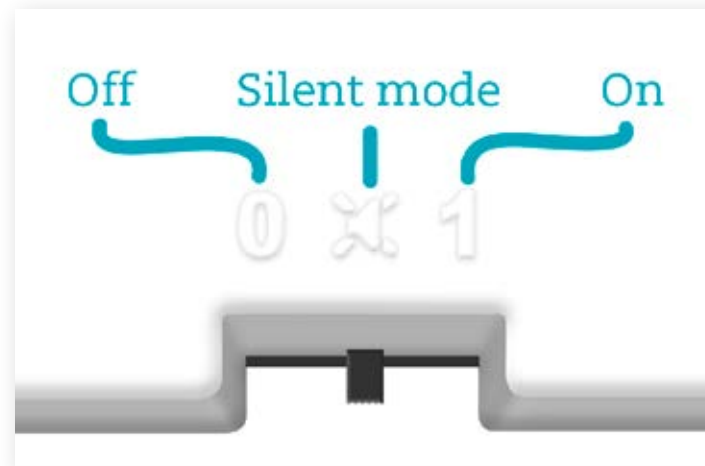Something not working out? We will gladly take you by the hand through the chat on our website or by phone.

## CHANGING THE WI-FI NETWORK

### Disconnecting the receiver

- Make sure you have your Wi-Fi password available.
  - Go to 'My Wizzu' through the burger menu in the top left corner of the app.
  - Select the receiver of which you want to change the Wi-Fi network and press 'delete'.
- 

### Selecting a new Wi-Fi network

- Press 'pair' Wizzu'.
- Turn the receiver off.
- Only turn on the receiver when the app says to do so.
- You hear a noise, this is the sign that it is working and the battery is charged.
- Connect to the "Wizzu" Wi-Fi network in the app.
- The app searches for nearby Wi-Fi networks.
- Select your home network and enter the password.
- When the receiver is connected correctly, you will hear a success sound.

## THE EVENING RITUAL

The moment before bedtime is very important while using the bedwetting alarm. A moment of rest and awareness.

1. Let your child hear the alarm sound a few times every night before going to sleep. This way your child knows when to wake up to.
2. Let your child go to the toilet before he/she goes to sleep.
3. Let your child put on the Dryly pants and press the transmitter in place.
4. Check the app with your child.
5. Read a book or let your child read for at least 20 minutes.
6. Let your child go to the toilet again.
7. Wish your child good night and put Wizzu about 1 meter away. This way your child has to get up and really wake up to turn off the alarm.

Sweet dreams!

## DRYLY WEBSHOP

### Pants

The Dryly pants are made of organic cotton and have a perfect fit. We developed the cheerful prints with children.

The bottoms can be washed at 60 degrees Celsius; they can also be put in the dryer. Avoid the use of fabric softener or chlorine. We also have neutral bottoms for children who are a bit older or for sleepovers.

### Mattress protector

Place the mattress protectors under the fitted sheet to protect your mattress. This prevents stains and unpleasant odors.

### Wizzu

Wizzu is the sweet panda, a great friend for your child. You can always order a Wizzu for a brother or sister.

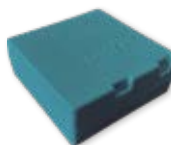
### The transmitter

The transmitter detects the moisture in the Dryly underpants. As soon as an accident occurs, a signal is sent to the receiver. Do not wash the transmitter because it will not work.

### The receiver

The receiver is placed in Wizzu and makes a wake-up sound in the event of an accident. The receiver immediately sends a message to the mobile app so that you can help your child. Parental support is crucial to the success of this method. In the app you can see the state of the battery. Experience shows that you need to charge the receiver once a week.

## ERROR MESSAGES

### ● Error code 1

This happens if the receiver tries to connect to the specified home network unsuccessfully. The network is within range but does not provide access to the receiver.

#### Cause

This error code usually occurs if an incorrect password was entered during pairing. If the receiver worked as planned for a bit gives this error message, it is possible that someone from the household has changed the password of your home network.

#### Solution

You solve this error message by reconnecting the receiver to the home network.

### ● Error code 2

You will get this message if the receiver has successfully connected to your home network but doesn't seem to be connected to the internet.

#### Cause

This error code can have multiple causes. The most obvious one is that your home network does not have an internet connection. Another common cause of this message is the use of 'Wi-Fi extenders'. In many households these are used to amplify the Wi-Fi signal so that there is also range in the basement or attic. If the 'Wi-Fi extender' itself has a bad connection with your Wi-Fi network, the receiver will obviously not be able to connect to the internet through this device.

*Solution*

You can easily test the quality of the connection by connecting to the same Wi-Fi network that the receiver is connected to. Then go to a website or YouTube and see if the content loads quickly.

If the problem persists, you can try to connect to another network if available.

---

**TIP!**

Unsure whether the network is the problem?

Create a hotspot with a phone and connect the receiver to it. If this works, you'll know it's on your network.

---

● **Error code 3**

This notification will appear if the recipient successfully connects to the internet, but does not get a connection to our secure servers. *If you get this error message it is a good idea to contact us.*

## ERROR MESSAGES

● **Transmitter out of range**

It can sometimes happen that you hear the message 'transmitter out of range'. This means that the receiver has been searching for the transmitter's signal for a while, but has not been able to detect it.

*Cause*

There are several possible causes for this message. If you also hear the message 'transmitter back in range' again, you probably have a less than stellar connection:

*Your child sleeps on his or her stomach*

The body blocks signals from the transmitter. If your child sleeps on his or her stomach, this can cause the signals not to get as far. You improve the connection by placing the receiver at the hip of your child's body. For example, you can put a chair next to the bed with Wizzu on it.

*The transmitter is not yet connected to the receiver*

When you receive our bedwetting alarm set, pairing the transmitter is part of the process. It may be that this step did not go quite right. This cause is only possible if you have never received an alarm before. Have you received an alarm already? Then the transmitter is paired and you have to look for another cause.

*The transmitter is too far away from the receiver*

The transmitter works on one battery for at least 6 months! We have been able to achieve this by making it extremely energy-efficient. The signal sent from the transmitter is therefore not strong enough to go through walls and doors. As a rule of thumb, keep the transmitter no further than 2 to 3 meters from the receiver. This will increase the quality of the connection.

*The battery is depleted*

If you have had the transmitter in your possession for a while, it is possible that the battery is depleted. On our website at frequently asked questions you will find an explanation about how to change the battery.

## WARRANTY

Dryly is committed to good quality products on a daily basis. It may happen that your Dryly bedwetting alarm does not work properly. Dryly b.v. complies with the law on market practices and consumer protection.

Dryly has a warranty period of 6 months for electronics and 3 months for underwear upon receipt of the bedwetting alarm by the purchaser. The warranty covers manufacturing and/or material defects and defects caused by normal use. This is at Dryly's discretion.

If your bedwetting alarm does not work as it should, follow these steps;

Report your complaint, preferably by email to [info@dryly.com](mailto:info@dryly.com), with the subject; 'complaint' with your order number. We will respond as soon as possible and do everything we can to solve your problem.

Damage caused by careless handling or by not following the instructions as stated in the user manual is excluded. The warranty can be invoked upon presentation of the invoice. The warranty is invalidated if repairs have been carried out by third parties, if the user has worn a nappy over the pants with the transmitter, if the bedwetting alarm has been purchased through a reseller (e.g. Marktplaats or eBay) and if water damage has occurred.

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